

Patient Application Instructions

BEFORE USING THE DEVICE, PLEASE READ THE INSTRUCTIONS FOR USE COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

SIZING INFORMATION

PART NUMBER	PATIENT HEIGHT	BRACE HEIGHT	SIZE
82-0052-1	≤1.5 m	17 cm	S
82-0052-2	1.5 m < x ≤1.65 m	23 cm	M
82-0052-3	>1.65 m	25.5 cm	L

The product is designed to be worn during normal everyday activities and sports. It can be worn over a sock. For additional support and optimal stabilization, we recommend to wear a closed shoe with the brace.




1. Place the liner pads inside the rigid bilateral shells.



2. Adjust the heel strap for proper fit.



3. Pull the outer shells and foam liner along the ankle, ensuring the malleoli are well protected.

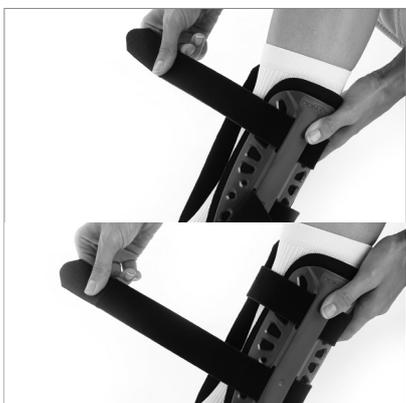

4. Make sure the brace fits snugly, and then close the brace using the contact closures.



5. Adjust the heel pad with the vertical side straps while the brace is on if necessary. Pull straps upward until the desired fit is obtained.



6. For maximum comfort and support, wear a long absorbent sock and a laced athletic shoe when wearing the brace.

Patient Application Instructions

INTENDED USE / INDICATIONS:

The FusioLight II® is designed to support the ankle for mild to moderate ankle sprains and help with chronic ankle instabilities (CAI).

- Acute lateral ligament injuries grade 1 & 2
- Conservative treatment of ligament injuries
- Initial management of chronic ankle instabilities
- Prevention of re-injury

WARNINGS AND PRECAUTIONS

Please see Instructions for Use for a full list of contraindications, warnings, and precautions.

CLEANING INSTRUCTIONS

To clean liner, hand wash in cold water using mild soap. Air dry

INTENDED FOR SINGLE PATIENT USE

NOT MADE WITH NATURAL RUBBER LATEX

enovis™

Enovis
1a Guildford Business Park, Guildford, Surrey, GU2 8XG, UNITED KINGDOM

enovis-medtech.eu

Copyright © 2022 DJO, LLC, a subsidiary of Enovis Corporation
MKT-000-00090004-EN - Rev A

Individual results may vary. Neither Enovis, DJO, LLC or any of their subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you.