

enovis™

DONJOY®
OA KNEE BRACES



DONJOY®

A legacy of knee bracing

DonJoy® has been supporting knees since 1978. From those first prototypes invented in the garage of our founders, to the products forged in our factories today, the company's legacy covers a multitude of braces, which in turn have covered a multitude of knees.

For more than 40 years, addressing knee osteoarthritis (OA) has been a constant mission for our product development teams. Their responses to this challenge have produced some of the best-known names in the world of knee OA bracing.

In the early 1990s, DonJoy introduced the Monarch. It marked the new millennium with OA Adjuster. And OA Nano became the world's lightest OA brace when it launched in 2012.

Each development advanced DonJoy's reputation as a trusted supplier of OA knee bracing solutions, and in turn improved the experiences of knee OA patients worldwide, helping them regain the joy of natural motion.

With the addition of ROAM™ OA and OA GO to its portfolio, DonJoy reaffirms its mission to improve the lives of OA sufferers by providing innovative, effective OA knee bracing.



Knee braces for every stage of osteoarthritis

Our portfolio of knee OA braces covers every stage of OA severity. Whether a patient has early-onset or severe knee osteoarthritis, DonJoy® has the brace to help relieve their knee pain and get them moving again.



Please consult your healthcare professional prior to use

ROAM™ OA

The newest innovation in offloader braces from DonJoy®

Help patients say goodbye to knee pain and hello to ROAM™ OA. From picking up their grandkids to teeing off at the driving range, lightweight, low-profile joint offloading and support never felt so good—or was so easy to prescribe. ROAM helps improve mobility and provide relief by offloading the pressure of moderate to severe unicompartmental osteoarthritis or other knee pain¹.

ROAM OA is ideal for a wide variety of patients who want to increase activity and who enjoy an active lifestyle. For conservative care or in preparation for surgery, ROAM is easy to use and offers a high level of comfort.

INTENDED USE/INDICATIONS

- Moderate to severe unicompartmental knee osteoarthritis
- Post-operative meniscal tears (only where unicompartmental unloading is indicated)
- Varus-valgus knee conditions that may benefit from unicompartmental unloading


ORDERING INFORMATION

PART NUMBER		THIGH CIRC.	KNEE CENTER	CALF CIRC.	SIZE
RIGHT MEDIAL/ LEFT LATERAL	LEFT MEDIAL/ RIGHT LATERAL				
11-1726-1-06000	11-1725-1-06000	33 - 47 cm	30.5 - 35.5 cm	25.4 - 35.5 cm	XS/S
11-1726-3-06000	11-1725-3-06000	47 - 59.7 cm	35.5 - 43.2 cm	35.5 - 45.7 cm	M/L
11-1726-5-06000	11-1725-5-06000	59.7 - 81.3 cm	43.2 - 58.4 cm	45.7 - 61 cm	XL/XXL/XXXL


ROAM OA can be fitted for either left lateral and right medial, or right lateral and left medial, for a drastically reduced SKU count.

Also available in custom-made version

1. Ramsey DK, Russell ME. Unloader Braces for Medial Compartment Knee Osteoarthritis: Implications on Mediating Progression. *Sports Health - A Multidisciplinary Approach*. 2009;1(5):416-426.

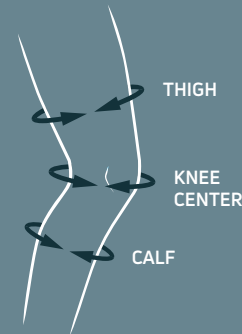


FEATURES AND BENEFITS

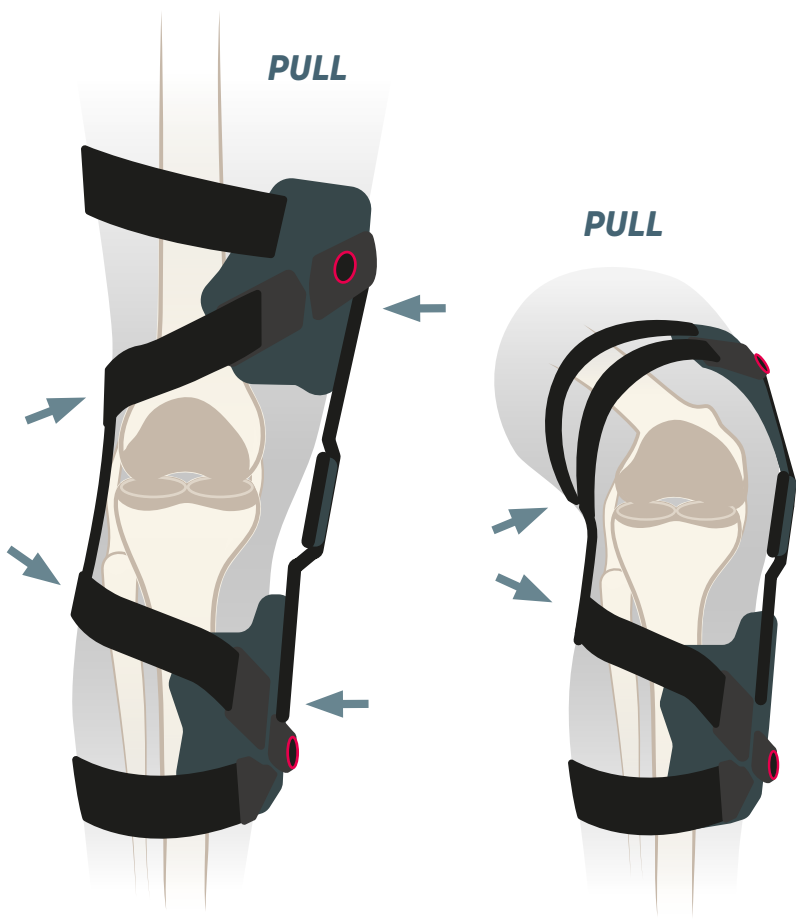


DYNAMIC OFFLOADING:

As the knee moves towards full extension or flexion, ROAM™ OA's condyle pad and harness increase or decrease in tension respectively. This focuses offloading when needed most, and increases comfort.

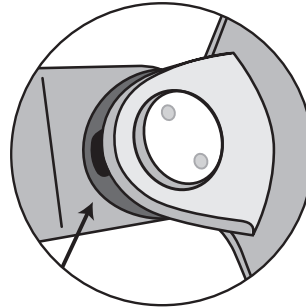


EASY MAG-ALIGN CLIPS:

These patented magnetic clips help align the strap connections without any fuss. Just hold the end near the attachment cap and 'click', you're set.

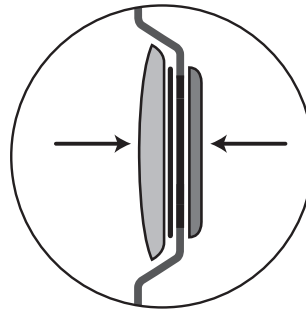


SLIM PROFILE HINGE:

The slim hinge design allows the brace to be worn under relaxed clothing for a discrete fit.

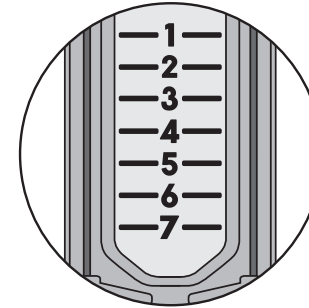


SET AND FORGET TECHNOLOGY:

You can don and remove your brace without changing any of your doctor's settings.

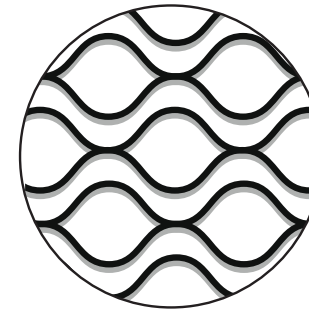


ANTI-MIGRATION TECHNOLOGY:

Soft silicone helps keep the brace in place for steady comfort throughout the day.

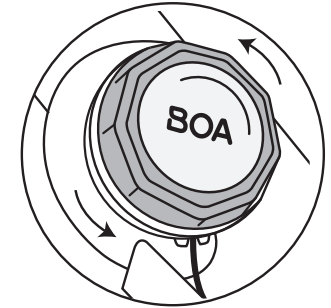

BOA® DIAL FOR JUST THE RIGHT FIT:

Dial up or dial down to set exactly how snug the brace fits.

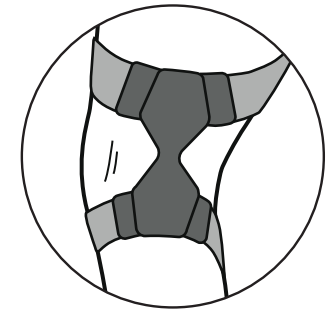

PATENT-PENDING COMFORT STRAPS:

The positioning of the support straps are engineered to help the back of your knee remain free from strap rub.

DONJOY OA GO®

Pain relief with a twist.

OA GO from DonJoy® helps provide fast and easy relief from pain and mobility issues caused by mild to moderate knee osteoarthritis². With a simple twist of a dial, this innovative soft knee brace's three-point leverage system quickly offloads the affected compartment to help relieve pain and ease movement.

OA GO is a low profile, lightweight combination of sleeve and wrap that slides over the leg for easy application. Its flexible hinge connects to a system of cables controlled by dual BOA® dials—just turn to achieve the necessary level of offloading and you're ready to go.

INTENDED USE/INDICATIONS

- Mild to moderate single compartment osteoarthritis (medial or lateral)
- General instability of the knee joint by age
- Varus-valgus knee conditions that may benefit from unicompartment unloading


ORDERING INFORMATION

PART NUMBER		THIGH CIRC.	SIZE
RIGHT MEDIAL/ LEFT LATERAL	LEFT MEDIAL/ RIGHT LATERAL		
82-7428-1	82-7429-1	33 – 39 cm	XS
82-7428-2	82-7429-2	39 – 47 cm	S
82-7428-3	82-7429-3	47 – 53 cm	M
82-7428-4	82-7429-4	53 – 60 cm	L
82-7428-5	82-7429-5	60 – 67 cm	XL
82-7428-6	82-7429-6	67 – 75 cm	XXL




2. Mistry DA, Chandratreya A, Lee PYF. An Update on Unloading Knee Braces in the Treatment of Unicompartmental Knee Osteoarthritis from the Last 10 Years: A Literature Review. Surg J (N Y). 2018 Jul 2;4(3):e110-e118.

FEATURES AND BENEFITS

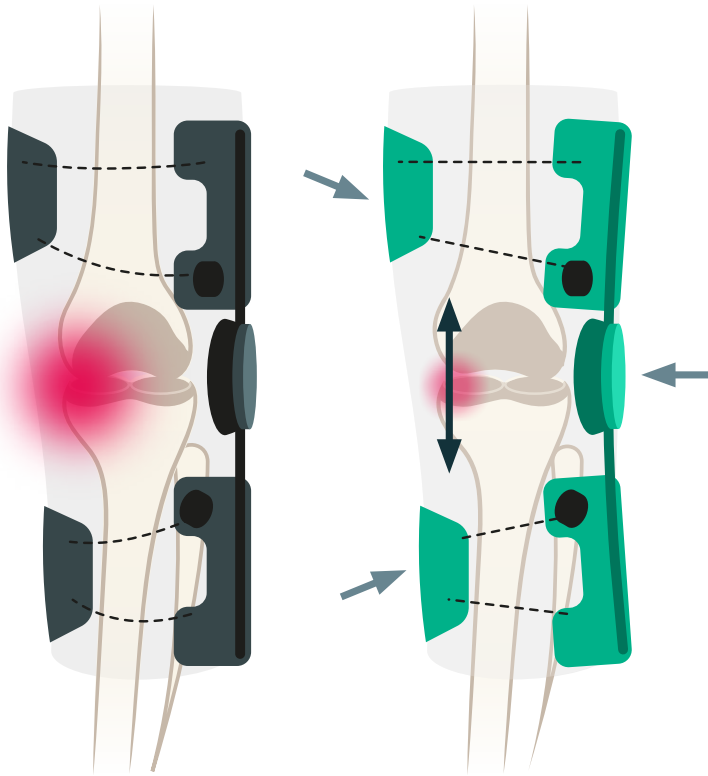


UNIQUE 3-POINT LEVERAGE SYSTEM

OA GO's patented fitting system uses adjustable BOA® dials to control the level of tension within its two cable loops. This tension reorients the flexible elements of the brace to create a three-point leverage system that offloads the affected compartment and helps provide relief.

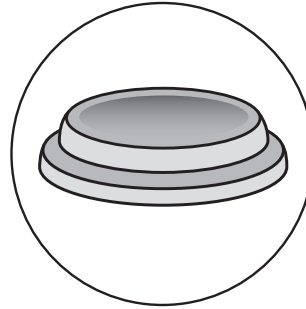


ANATOMICALLY CONTOURED CONDYLE PADS:

Adhere to the knee anatomy for improved pressure distribution. Two different thicknesses (10 mm and 13 mm) of condyle pad included to aid individual fit and comfort for varied anatomies.

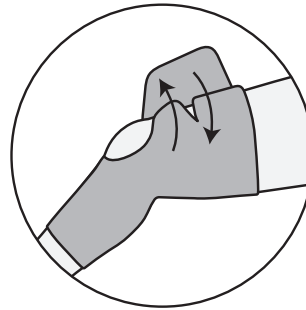


HYBRID SLEEVE/WRAP DESIGN:

Half wrap, half sleeve, OA GO combines the benefits of both to make fitting quick and easy. Simply open the thigh section then pull the lower sleeve section over the calf before securing the support with the hook-and-loop fastening.

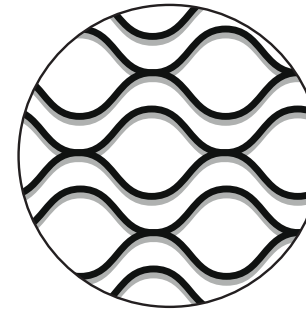


ANTI-SLIP BANDS:

Positioned at the top of the thigh to prevent brace migration and maintain a comfortable fit.

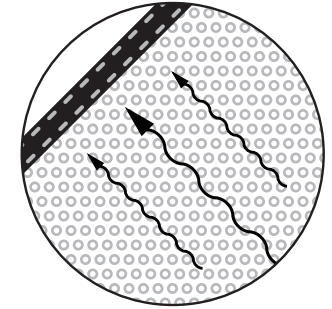


BREATHABLE, STRETCHABLE, ANTIBACTERIAL FABRICS:

Aid comfort, compliance, and hygiene, enabling extended periods of use. Not made with natural rubber latex. Rear lycra section provides additional comfort and freedom over the popliteal fossa.

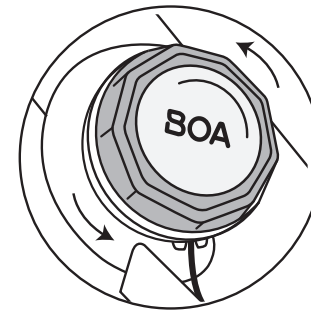

PRECISION FIT AND FAST RELEASE:

Twin multidirectional BOA Li2 dials deliver a precision fit engineered to perform in the toughest conditions. Turn clockwise and counter-clockwise for independent tightening and releasing of thigh and calf areas, or pull up for quick release.

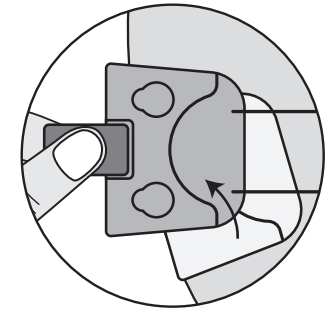

EASY-GRIP PULL TAB:

Facilitates quick and easy attachment and release of the locking plate, and folds down flat to avoid catching.

DONJOY®

Asia Pacific

1905, Tower 2, Grand Central Plaza
Shatin
HONG KONG
T: 852 31051415
E: marketing.ap@enovis.com

Australia – New Zealand

PO Box 6057
Frenchs Forest DC
NSW 2086
AUSTRALIA
T: 1300 66 77 30
E: customerservice.au@enovis.com

Benelux

Kleinhoefstraat 5, bus 39
Geel 2440
BELGIUM
T: +32 (0)14 248090
E: orders.bx@enovis.com

Canada

6485 Kennedy Rd
Mississauga
Ontario, L5T2W4
CANADA
T: 1-866-866-5031
E: canada.orders@enovis.com

China

A312 SOHO ZhongShan Plaza
1055 W. ZhongShan Rd
Shanghai, 200051
CHINA
T: (8621) 6031 9989
E: shujun.tang@enovis.com

France

3 rue de Bethar
Centre Européen de Frêt
64990 Mouguerre
FRANCE
T: +33 (0)5 59 52 86 90
E: sce.cial@enovis.com

Germany

Bötzingler Straße 90
79111 Freiburg
GERMANY
T: +49 (0) 180 1 676 333
E: kundenservice@enovis.com

Italy

Via Leonardo da Vinci 97
Trezzano sul Naviglio
Milan 20090
ITALY
T: +39 02 835 98001
E: vendite@enovis.com

Nordic

Murmansgatan 126
212 25 Malmö
SWEDEN
T: +46 40 39 40 00
E: info.nordic@enovis.com

South Africa

Unit 6, 2 on London, 2 London Circle
BrackenGate Business Park
Brackenfell, 7560
SOUTH AFRICA
T: +27 (0) 21 276 2968
E: info.southafrica@enovis.com

Spain

Av.Cornella 144. 1-4
08950 Esplugues de Llobregat
Barcelona
SPAIN
T: +34 93 480 32 02
E: ventas@enovis.com

UK & Ireland

1a Guildford Business Park
Guildford, Surrey
GU2 8XG
UK
T: +44 (0)1483 459 659
E: ukorders@enovis.com

USA

2900 Lake Vista Drive
Lewisville
TX 75067
U.S.A.
T: 760 727 1280
E: customercare@enovis.com

enovis™

Creating Better Together™

enovis-medtech.eu

1a Guildford Business Park, Guildford, Surrey, GU2 8XG, UNITED KINGDOM

Individual results may vary. Neither Enovis, DJO, LLC or any of their subsidiaries dispense medical advice. The contents of this document do not constitute medical, legal, or any other type of professional advice. Rather, please consult your healthcare professional for information on the courses of treatment, if any, which may be appropriate for you. BOA® is a registered trademark of BOA® Technology Inc.

