## ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

### INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. The user should be able to read, understand and be physically capable to perform all the directions, warnings and cautions provided in the information for use

### INTENDED USE/INDICATIONS:

The DonJoy Strapping Elastic Ankle is designed to provide support to the ankle following injuries such as ligament damage and chronic ankle instabilities. Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

**PERFORMANCE CHARACTERISTICS:** Soft-good/semi-rigid designed to restrict motion through elastic or semi-rigid construction.

### CONTRAINDICATIONS:

N/A

### APPLICATION INSTRUCTIONS:

- 1) Put on an elastic sock.
- 2) Form a loop with the bandage. Loop this over the elastic sock and close on the instep. (Fig. 1)
- 3) If you wish to reinforce the inside of the ankle, pull the bandage towards the outside. (Fig. 2)
- 4) If you wish to reinforce the outside of the ankle, pull the bandage towards the inside. (Fig. 3)
- 5) Wrap the bandage underneath the foot, holding the bandage tight. (Fig. 4)
- 6) Then wrap the bandage over the instep. (Fig. 5)
- 7) Wrap it around the foot, level with the Achilles tendon. (Fig. 6)
- 8) Wrap it over the instep again. (Fig. 7)
- 9) Pass it under the foot and attach the self-adhesive/velcro tab to the bandage itself. (Fig. 8)

The self-adhesive / velcro tab should be attached to the bandage. If it is upside-down, undo the bandage, reverse it and start again.

### CARE INSTRUCTIONS:

Wash in cold water. Do not bleach. Do not dryclean. Do not iron

### WARNINGS AND PRECAUTIONS:

- This device is to be prescribed and fitted by a healthcare professional.
- 1) Do not use over open wounds.
  - 2) Do not use this device if it was damaged and/or package has been opened.
  - 3) Do not use this device if it was damaged and/or package has been opened
  - 4) If you develop an allergic reaction and/or experience itchy, red skin after coming into contact with any part of this device, please stop using it and contact your healthcare professional immediately.

**NOTE:** Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

### FOR SINGLE PATIENT USE ONLY

**WARRANTY:** DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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