ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT SET-UP IS VITAL TO THE PROPER USE OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient, the patient's caretaker, or a family member providing assistance. User should be able to:

- Read and understand the directions, warnings and cautions.

OPERATING PRINCIPLE: The Iceman Bed Hanger supports the cold therapy unit.

INTENDED USE/INDICATIONS: Support of IceMan® cold therapy unit on bed or gurney.

1. In the upright position shown, hook the bed hanger at the foot of the bed. Secure the cooler unit on the bent area of the hanger. The prongs of the hanger should be pointing out.
2. By reversing the position of the bed hanger, it can be hooked to the end of a bed or on the side of a gurney. Secure the cooler unit on the bent area of the hanger. The prongs of the hanger should be used as hooks.

CONTRAINDICATIONS: None

CAUTIONS: DO NOT use if bent area of hanger is not completely and securely seated over edge/rail of gurney or bed. The cooler must also be held completely and securely on adjacent bent area of the hanger as shown above. Cooler must remain in upright position to prevent spilling of water and/or ice.

CLEANING INSTRUCTIONS: The hanger may be cleaned with mild detergent and warm water.

WARNINGS: THIS PRODUCT IS TO BE USED UNDER THE SUPERVISION OF A MEDICAL PROFESSIONAL. THIS DEVICE IS NOT INTENDED FOR UNSUPERVISED PUBLIC USE. IF YOU EXPERIENCE ANY PAIN, SWELLING, SENSATION CHANGES, OR ANY UNUSUAL REACTION WHILE USING THE COLD THERAPY DEVICE, PLEASE CONSULT YOUR MEDICAL PROFESSIONAL IMMEDIATELY. KEEP SMALL PARTS AND PACKAGING MATERIALS AWAY FROM SMALL CHILDREN AND ANIMALS. THESE ITEMS POSE A RISK FOR SUFFOCATION OR STRANGULATION.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale.

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