ENGLISH

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE:

The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable to perform all the directions, warnings and instructions provided in the use for use.

INTENDED USE/INDICATIONS: This DonJoy Knee Brace with articulation stops installed is designed to provide bracing support to the medial and lateral compartments of the knee. It is indicated for ACL/PCL deficiencies, collateral ligament deficiencies, collateral ligament deficiencies with PCL involvement and MCL and/or LCL sprains. Soft/semi-rigid design to restrict motion through a combination of elastic and semirigid. Provides a moderate level of protection through a semi-rigid construction. Providing mild protection of a limb or body segment for acute and prophylactic care.

CONTRAINDICATIONS: Do not use if you are allergic to any of the materials contained in this product.

WARNINGS AND PRECAUTIONS:

- Loss of circulation, patient discomfort and patient re-injury are potential effects caused by device failure.
- ROM settings should not be changed without supervision of a healthcare professional.
- This product must be prescribed by a healthcare professional.
- The frequency, duration and use of directions for use should be determined by your healthcare professional.
- If you experience changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- The support should be snug but not impair circulation.
- Do not use over open wounds.
- Do not use if the device it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

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INSTRUCTIONS FOR HINGE ADJUSTMENT:

- Your braces comes with 10° extension stops installed. To change the installed stops, first completely remove the screw as shown (for Armor by DonJoy & Legend (figure 5), for 4TITUDE Advanced & Renegade (figure 6)). Then align the holes in the hinge plate, the stop and the screw, align with the holes in the hinge plate, then reinsert the screw. Using a Phillips head screwdriver, ensure that the screws are seated in the recess and completely tightened. Both medial and lateral shield slide cover (cover on LEGEND).

USE AND CARE:

- If during activity, re-tightening is necessary, re-tighten all straps following steps in the application instructions.
- All straps and pads may be trimmed to fit individual variations in knee circumference.
- The brace may be used in fresh or salt water. Following use in water activities, drain thoroughly and rinse with clean, fresh water and then air dry.
- Hand wash your liners and reinforce pads in cold water with mild liquid soap. Dry with a soft cloth.
- The hinge mechanism requires little maintenance in normal use. However, you may apply a dry lubricant such as a Teflon® spray to the internal mechanism of the hinge. Dry lubrication is recommended following use in water.
- Periodically inspect hinge screws for tightness and retighten as needed.

CLEANING INSTRUCTIONS: Hand wash in water (86°F/30°C) using mild soap. Air dry.

MATERIAL CONTENTS: Frame- Aluminum & Nylon, Hinge- Aluminum & Steel, Strap pads - Aluminum & Nylon, Soft gels - Nylon & EVA.

WARNING: DJO, LLC will repair or replace any part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

Rx ONLY.
INTENDED FOR SINGLE PATIENT USE.

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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INSTRUCIONES DE COLOCACION:

- A: While sitting on the edge of a chair, step through the brace.
- Flexing the knee to 45°, coloque la rodilla como se muestra en la figura 1, centrándola sobre la rótula y un poco más allá de la línea media de la pierna. La rodilla debe quedar recta en la pierna. B: Secure the bottom calf strap first (#1), then the top thigh strap (#2).
- Keep the knee bent at 45 degrees, push both condyles (hinges) toward the back of the leg, as shown in (figure 2).
- Maintaining the position of the hinges: for ACL version secure lower thigh strap (#3) for PCL version secure upper calf strap (#4) on the back of the leg.
- For ACL version, draw the hinges forward slightly before fastening the front tibial (shin) strap (#4). For PCL version draw the front tibial strap slightly before the hinge strap (#4).
- Maintaining the position of the hinges: for CI version, secure the upper thigh strap (#5) for the PCL version secure the lower thigh strap (#6).
- When all strapping is complete, the hinge should center just above the top of the kneecap and slightly bent. To ensure that no strap occurs during activity, it is recommended to do a few knee bends or walk for a few minutes, then re-tighten the straps following steps B or E for CI version.

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