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DJO, LLC
5919 Sea Otter Place,
Suite 200
Carlsbad, CA 92010
USA

BEFORE USING THE DEVICE, PLEASE READ THE FOLLOWING INSTRUCTIONS COMPLETELY AND CAREFULLY. CORRECT APPLICATION IS VITAL TO THE PROPER FUNCTIONING OF THE DEVICE.

INTENDED USER PROFILE: The intended user should be a licensed medical professional, the patient or the patient's caregiver. The user should be able to read, understand and be physically capable of performing the directions, warnings and cautions in the information for use.

INTENDED USE/INDICATIONS: • Elbow hyper extension
• Ligament injuries of the elbow (UCL) • Radial head fractures

APPLICATION INFORMATION:

A) After pulling brace on affected arm, first close bicep flat to desired tightness. (Figure 1)

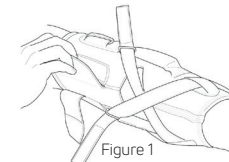
Figure 1

B) Secure bicep strap. Strap does not need to be very tight. (Figure 2)

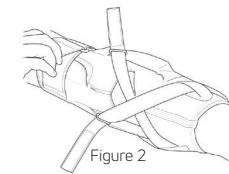
Figure 2

C) Secure tricep strap. This rigid strap needs to be secured tightly. (Figure 3) Do not tighten so tight as to impair circulation.

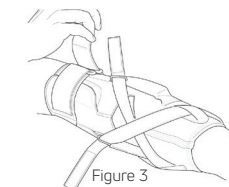
Figure 3

D) Bend elbow to desired flexion angle and secure first cross strap. (Figure 4)

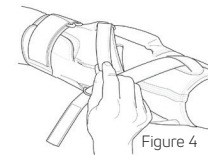
Figure 4

E) With elbow still bent, secure last cross strap. Before activity, check to make sure all straps are secure. (Figure 5)

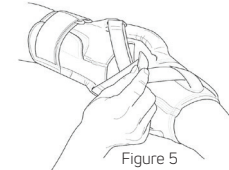
Figure 5

CLEANING INSTRUCTIONS: Hand wash in cold water with mild soap, rinse thoroughly. AIR DRY. Note: If not rinsed thoroughly, residual soap may cause irritation or deteriorate material.

WARRANTY: DJO, LLC will repair or replace all or part of the unit and its accessories for material or workmanship defects for a period of six months from the date of sale. To the extent the terms of this warranty are inconsistent with local regulations, the provisions of such local regulations will apply.

WARNINGS AND PRECAUTIONS:

- If you have skin disorders/injuries in the relevant part of the body, the product should be fitted and worn only after consultation with a medical professional.
- If pain, swelling, changes in sensation or other unusual reactions occur while using this product, you should contact your doctor immediately.
- The support should be snug but not impair circulation.
- Do not use over open wounds.
- Do not use this device if it is damaged and/or the packaging has been opened.

NOTE: Contact manufacturer and competent authority in case of a serious incident arising due to usage of this device.

FOR SINGLE PATIENT USE ONLY

NOTICE: WHILE EVERY EFFORT HAS BEEN MADE IN STATE-OF-THE-ART TECHNIQUES TO OBTAIN THE MAXIMUM COMPATIBILITY OF FUNCTION, STRENGTH, DURABILITY AND COMFORT, THERE IS NO GUARANTEE THAT INJURY WILL BE PREVENTED THROUGH THE USE OF THIS PRODUCT.

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